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**English**  
**English:**

**medi clean**  
**Handwash detergent**  
Formulated for compression stockings and bandages

Washing your compression stockings every day is as vital for your comfort as having the correct fit. With medi clean, you can gently clean your stockings whilst retaining their weft and colour.

The odourless handwash detergent is dermatologically approved.

**Usage Instructions**  
Mix 10 ml with 2-3 litres of water for one pair of stockings.

Water temperature: Lukewarm

**medi spot ex**  
**Spot cleaner**  
Specially formulated for compression stockings and bandages  
For removing marks caused by shoes. Apply before daily washing.

**Usage Instructions**  
Apply the spot cleaner to the mark, massage it in gently using your fingertips and leave it to work for around 5 minutes.

Rinse the fluid out of the aid under running, lukewarm water.

Then wash and dry the compression stocking according to the instructions for use or the label.

**General Instructions**  
Your medical compression stockings from medi are suitable for both machine and handwashing as well as the tumble dryer. You can see this as it is stated on the garment's label, using the illustrated symbols.

Caring for your stockings is a part of actively ensuring optimal therapy with medical compression stockings. Washing your stockings daily ensures that the fabric remains breathable and fits correctly.

medi clean and medi spot ex are odour-neutral and dermatologically approved. The colour and fabric of your stockings will be gently cleaned for daily use.

All medi skin and fabric products are designed to work optimally in combination with each other. This ensures that wearing medi products is comfortable, every day!

medi clean and medi spot ex are not subject to any minimum storage life or best before dates. Therefore you will not find any information on minimum storage life on the product packaging or this instruction leaflet.

**Health warning**  
To protect your health, there are warning indications on the products' characteristics as well as handling instructions and telephone numbers which are to be used in the event of an emergency.

medi clean is a concentrate! This means that you must only use the handwash agent once it has been diluted, in a mixture where there is 10 ml of agent to 2-3 litres of water, for example.

If the undiluted liquid comes into contact with the eye, this may cause irritation.

Rinse your eyes thoroughly with water straight away and consult your doctor.

If any of the undiluted liquid is swallowed, obtain medical advice immediately and show the original professional the original product.

This handwash agent must be kept out of reach of children.

**Remarks**  
Similar instructions are given for standard handwash agents for crockery, for example.  
The form and scope of these details are set down as requirements by law and valid throughout the European Union.

**Disposal**  
Please dispose of the completely empty bottle in accordance with the guidelines at a collection point for plastics.  
There are no specific criteria for disposal as hazardous waste.

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