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EN: Adhesive lotion for securing medical compression stockings and elastic compression garments

General Instructions

General Instructions: Please read through this information carefully. You will receive valuable tips on application, care and disposal. Keep the instructions for use. You may wish to read them again at a later point in time. If you have any questions, please contact the doctor treating you, the medical supply store or the medi Consumer Service.

Compression therapy aids include, among other items, medical compression stockings which are manufactured in different lengths, compression classes and with different knitting methods. Compression stockings are used for a variety of purposes, including* treating venous disorders and conditions and for treating oedema caused by lymphoedema and lipooedema.
* For indications for medical compression stockings see medi and/or mediven instructions for use.

Important general instructions

- This product is not self-igniting.
- Please do not allow this substance to come into contact with your eye. Should some of the substance get into your eye, rinse your eye with running water.
- Keep out of reach of children.
- Keep away from sources of ignition.

Important instructions for use

- Use only on unbroken skin, legs and arms.
- If you find that you are allergic to any of the ingredients of medi fix then please remove the stocking and clean the affected areas gently and carefully. If necessary, visit a medical specialist retailer for advice on an alternative method of securing your stockings.
- If the area that you will adhere your stocking to is very hairy, then you should shave or otherwise remove the hair prior to use.
- Please note the following: If you apply the adhesive lotion directly

to your compression stocking then it will not be evenly applied and will form blobs and drips. Therefore please follow the user instructions correctly.
• Even just a thin layer of medi fix secures stockings well.

Storage instructions

- Please fully screw the top back onto the roller after it has been used. This will prevent the lotion from drying out.
- Ensure that you always stand the bottle on its base and not upside-down.
- Store in a cool place.

Instruction booklet

Usage

- Put on your compression stockings and ensure that they are positioned correctly.
- Fold down the band or knitted edge.
- Apply a thin, even layer of medi fix to your skin.
- Pull the upper part of the stocking back up and press the fabric against your skin, where the adhesive is.
- The stocking will now stay securely in position throughout the day without slipping.

Care and washing instructions

Cleaning

medi fix does not stain and can be washed off skin easily with water and soap. Please wash and dry the compression stockings in line with the instructions on the label or the instructions for use.

Disposal

Please dispose of the completely empty medi fix bottle in accordance with the guidelines at a collection point for plastics. There are no specific criteria for disposal as hazardous waste.

Product life

You can use medi fix adhesive lotion on a daily basis, as required by the therapy with medical compression stockings.

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