



medi

New development

medi Rehab one

Effective oedema therapy after
orthopaedic procedures and injuries
to the musculoskeletal system

medi. I feel better.

A world first by medi

Specially designed for posttraumatic and postoperative oedema therapy

medi Rehab one is the first medical compression stocking specially developed by medi to treat posttraumatic and postoperative oedema in the field of orthopaedics. It combines tried-and-tested medical compression with a new, knit technology and smart details. This makes using the stockings easier, safer and more convenient for your patients.

The medi Rehab one provides support for oedema therapy after many orthopaedic procedures and injuries, for example after ligament injuries in the knee or ankle, for fractures and also in the rehabilitation phase after arthroplasty procedures.



The medi Rehab one is a CCL 1 medical compression stocking. It has a defined graded pressure distribution from distal to proximal. Thanks to pressure on the tissue from the outside, existing oedemas are reduced and oedema formation is counteracted.

Easy to put on and a secure fit

The innovative micro-plush technology ensures that the medical compression stockings glide over the skin particularly well when donning the stockings. This is a major advantage, particularly for patients who are sensitive to pain, because using the stockings can be made significantly easier thanks to this new technology. At the same time, the micro-plush technology helps to ensure that the garment fits securely on the leg without slipping, even in rehabilitation or for sport. Sensitive topbands and cuffs provide additional grip.

Correct positioning

The medi Rehab one is impressive thanks to additional features, for example the 3D longitudinal ribs and a positioning aid at the knee. These details also make it easier for patients who are not used to wearing the stockings to don them correctly and provide more security in everyday use.

High level of user comfort

The micro-plush technology is particularly soft and pleasant to wear against the skin. The stocking has been equipped with special functional zones which add extra user comfort. Extra-thin and breathable in the sensitive knee and thigh area, particularly soft and featuring thermoregulating merino plush in the foot area.

The medi Rehab one is impressive thanks to lots of details and sets new standards in posttraumatic and postoperative compression therapy for oedema reduction and prevention.

An overview of the advantages

New, knit technology with micro-plush, which makes the stockings easy to put on and provides a secure fit

Positioning aid and 3D longitudinal ribs for secure use

Thin and especially breathable in the sensitive bending and thigh area

Merino plush in the midfoot and forefoot area for increased user comfort


Tried-and-tested CCL 1 medical compression.
Additionally: Comfortable compression from the midfoot area



Micro-grip top band for additional hold and to provide comfort on the skin.



Thin, very breathable, active material on the thigh and in the sensitive bending area at the hollow of the knee for additional comfort.

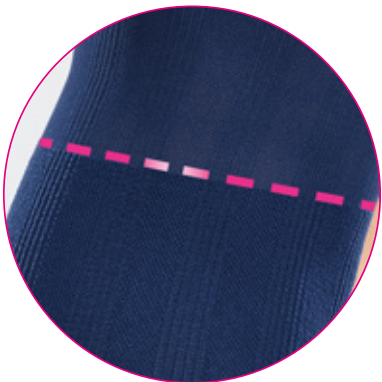

Positioning aid for easy and secure everyday use.



New, knit technology with micro-plush:

ensures that the medical compression stockings glide over the skin particularly easily when donning and fit securely on the leg.

Additionally: Merino plush in the midfoot and forefoot area for increased user comfort and thermoregulation
Tried-and-tested CCL 1 medical compression

Additionally: Comfortable compression from the midfoot area



Broad knee stocking band for additional hold.



3D lengthwise ribbing facilitates correct placement and provides secure handling.



Pre-formed heel for easy positioning and improved fit.



Forefoot is sized individually to provide improved fit.



All information at a glance

Intended Purpose

medi Rehab one is a round knit medical compression stocking for providing compression support for the lower extremities, primarily for the prevention and treatment of postoperative and posttraumatic oedemas and general swelling issues.

Main indications

Oedema

- Posttraumatic oedema
- Postoperative oedema

Why choose medi Rehab one?

- Knee sock (AD): Micro-plush makes the garment easy to put on
- Thigh-length stocking (AG): Positioning aid on the knee for easy and correct use


Donning videos



Use the QR code or short URL medi.biz/rehab-one to watch the donning videos.

How to measure correctly

ℓ = Length
c = Circumference

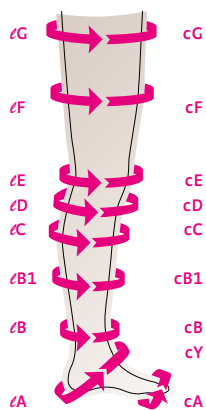

	Circumference in cm				
	II	III	IV	V	VI
Topband normal	45–52	49–56	53–60	56–64	60–68
Topband wide	53–62	57–67	61–72	65–77	69–81
cG	45–62	49–67	53–72	56–77	60–81
cF	41–56	44–60	47–65	50–69	53–73
cE	33–40	35–43	37–45	39–48	41–51
cD	29–36	32–39	34–42	36–45	38–48
cC	30–37	33–40	35–43	37–46	39–49
cB1	24–29	26–32	29–35	31–37	33–39
cB	20–22	22–24	24–26	26–28	28–30
cY	28–33	29–35	31–37	32–38	33–40
cA	19–24	21–26	23–29	25–32	27–34

Length in cm	Art.- no.	Version
Knee stocking (AD)		
39–44	RA40412–6*	normal
34–38	RA47412–6*	short
Thigh-length stocking (AG)		
72–83	RA61402–6*	normal silicone topband
	RA65402–6*	normal silicone topband -W
62–71	RA51402–6*	short silicone topband
	RA55402–6*	short silicone topband -W

* Select the size using the last digit of the item number from 2 to 6.

Headquarters
medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
P +49 921 912-0
F +49 921 912-783
export@medi.de
www.medi.de/en