**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

**medi Austria GmbH**  
**Adamgasse 16/7**  
**6020 Innsbruck**  
**Austria**  
**T +43 512 57 95 15**  
**F +43 512 57 95 15 45**  
**vertrieb@medi-austria.at**  
**www.medi-austria.at**

**medi Belgium bvba**  
**Posthoornstraat 13/1**  
**3582 Koersel**  
**Belgium**  
**T +32 011 24 25 60**  
**F +32 011 24 25 64**  
**info@medibelgium.be**  
**www.medibelgium.be**

**medi Brasil**  
**Rua Neusa 216**  
**Diadema-Sao Paulo**  
**Cep 09941-640**  
**Brazil**  
**T +55 11 3201 1188**  
**F +55 11 3201 1185**  
**sac@medibrasil.com**  
**www.medibrasil.com**

**medi Danmark ApS**  
**Vejlegardsvej 59**  
**2665 Vallensbæk Strand**  
**Denmark**  
**T +45 46 55 75 69**  
**F +45 70 25 56 20**  
**kundeservice@sw.dk**  
**www.meditandmark.dk**

**medi Bayreuth España SL**  
**C/Canigó 2 – 6 bajos**  
**Hospitalet de Llobregat**  
**08901 Barcelona**  
**Spain**  
**T +34 93 260 04 00**  
**F +34 93 260 23 14**  
**medi@mediespana.com**  
**www.mediespana.com**

**medi France**  
**Z.I. Charles de Gaulle**  
**25, rue Henri Farman**  
**93297 Tremblay en France Cedex**  
**France**  
**T +33 1 48 61 76 10**  
**F +33 1 49 63 33 05**  
**infos@medi-france.com**  
**www.medi-france.com**

**medi Hungary Kft.**  
**Bokor u. 21.**  
**1037 Budapest**  
**Hungary**  
**T +36 1371 0090**  
**F +36 1371 0091**  
**info@medi-hungary.hu**  
**www.medi-hungary.hu**

**medi Medical Support Sdn Bhd**  
**medi representative office Asia**  
**Unit No. B-2-19, Block B, No.2,**  
**Jalan PJU 1A/7A**  
**Oasis Ara Damansara, PJU 1A,**  
**47301 PETALING JAYA**  
**Selangor Darul Ehsan**  
**Malaysia**  
**T: +6 03 7832 3591**  
**F: +6 03 7832 3921**  
**info@medi-asia.com**  
**www.medi-asia.com**

**medi Middle East**  
**P.O. Box: 109307**  
**Abu Dhabi**  
**United Arab Emirates**  
**T +971 2 6429201**  
**F +971 2 6429070**  
**info@mediuae.ae**  
**www.mediuae.ae**

**medi**

## medi Collar soft

**Komfortable Halskrause**  
**Comfortable support for the cervical spine**  
**Collier cervical confortable**  
**Cómodo collarín**

Gebrauchsanweisung. Instructions for use. Mode d'emploi.  
 Instrucciones de uso. Instruções para aplicação. Gebruiksaanwijzing.  
 Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu.  
 Инструкция по использованию. Instrukcja zakładania. Οδηγία εφαρμογής.

E010624 / 07.2014

medi. I feel better.

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### Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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---

## Intended purpose

medi Collar soft is a cervical collar. The product is exclusively to be used for the orthotic fitting of the thoracic spine and only on unbroken skin.

---

## Indications

- Cervical syndrome
- Cervical arthrosis
- Torticollis
- Whiplash injuries
- Hypermobility
- Rheumatic problems

## Contraindications

None known at present.

## Mode of action

Stabilisation of the cervical spine with warming and supportive function. Limitation of the movement.

## Note on wearing

The anatomically shaped side is under the chin while the straight part rests on the collarbone.

---

## Care instructions

Velcro fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
  - Do not bleach.
  - Leave to dry naturally.
  - Do not iron.
  - Do not dry clean.
- 

## Storage instructions

Please keep the support in a cool, dry place and do not expose it to direct sunlight

## Material composition

PU foam, cotton

## Warranty / contractual guarantee

consistent with legal guidelines.

**Disposal**

The product can be disposed of in the domestic waste.

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