



medi UK Ltd.
Plough Lane
Hereford HR4 0EL
Great Britain
T: +44-1432 37 35 00
F: +44-1432 37 35 10
enquiries@medik.co.uk
www.medik.co.uk

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-781
export@medi.de
www.mediz/wound


medi

juxtacures®

Customising, donning and
setting the compression

medi. I feel better.

Customising

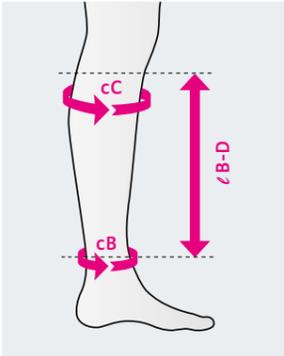

Step 1
First measure the length from floor to knee crease. Then take the circumference measurements of the ankle B and calf C.

Short (28 cm)	DT100/2
Standard (33 cm)	DT100/3
Long (38 cm)	DT100/4



Step 2
Take the spine piece and place the hook and loop fastener on the bottom edge of the body piece for ankle circumference B and on the top edge for calf circumference C.



Step 3
Fold over the spine piece and cut off the excess material leaving approximately 5 cm of overlap.

Donning



Step 1
First slide the circaid undersleeve onto the lower leg and over the knee. The cuffed edge of the undersleeve should cover the heel. Ensure that there are no wrinkles.



Step 2
Align the circaid juxtacures just above the ankle bone and below the knee crease. The black side faces the leg.



Step 3
Unroll the top two bands and secure them loosely onto the opposing side of the garment to hold the garment in place.



Step 4
Unroll the bottom two bands. Pull across the lowest band and attach it with firm and comfortable compression.



Step 5
Before attaching the second band, unroll the third band. Now pull across the second band and attach it.



Step 6
Repeat this step with the remaining bands. Inspect the garment for any gaps or creases. Readjust the bands if necessary.

Setting the compression

Step 7

Line up the triangle on the Built-In-Pressure System (BPS) card with one of the BPS-lines on the band. Note where the second line of the band lines up with the card's compression ranges. Adjust the compression to the prescribed compression range. Repeat this process for each band going up the leg.



Too less compression



Too much compression



Correct compression



Step 8
Once the compression is set, fold the undersleeve over the top and bottom edge of the garment. Finally, don the circaid compression ankle.



For more information and handling videos please visit:
www.medi.biz/juxtacures