

medi



juxtafit®

Donning and setting  
the compression  
for juxtafit lower leg

medi. I feel better.

# juxtafit<sup>®</sup>



## Donning



**Step 1**  
First slide the undersleeve onto the lower leg and over the knee. The cuffed edge of the undersleeve should cover the heel. Ensure that there are no wrinkles.



**Step 2**  
Align the juxtafit just above the ankle bone and below the knee crease. The black side faces the leg.  
  
Unroll the top two bands and secure them loosely onto the opposing side of the garment to hold the garment in place.



**Step 3**  
Unroll the bottom two bands. Pull across the lowest band and attach it with firm and comfortable compression.  
  
Before attaching the second band, unroll the third band. Now pull across the second band and attach it.



**Step 4**  
Repeat this step with the remaining bands. Attach the bands alternately from bottom to top so that the garment lays flat and wrinkle-free.



**Step 5**  
Inspect the garment for any gaps or creases. Readjust the bands if necessary.




**Step 6**  
Each band should be adjusted to the prescribed compression range. Use the provided card to identify the applied compression range.

## Setting the compression

### Step 7

Line up the triangle on the Built-In-Pressure System (BPS) card with one of the BPS-lines on the band. Note where the second line of the band lines up with the cards compression ranges. Adjust the compression to the prescribed compression range. Repeat this process for each band going up the leg.

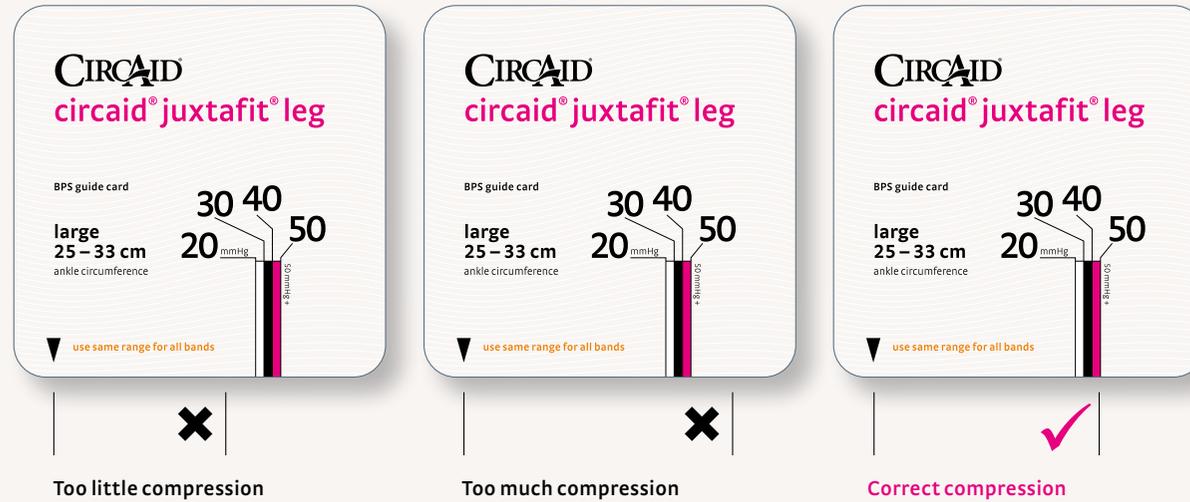

### Intended purpose juxtafit

The compression system is designed to provide compression to the leg for patients with venous and lymphatic disorders.



### Step 8

Once the compression is set, fold the undersleeve over the top and bottom edge of the garment. Finally, don the compression ankle.



For more information and handling videos please visit:  
[www.medi.biz/jf](http://www.medi.biz/jf)

medi. I feel better.

medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-781  
export@medi.de  
www.medi.biz/wound

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T: +44-1432 37 35 00  
F: +44-1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk