



UCS™ Debridement

A pre-moistened cleaning and debridement cloth

To use UCS:

1. Tear open the sterile UCS packet to access the cloth.
2. Use each area of the cloth to gently work into or around the wound to allow the fluid to penetrate the skin, enabling debris to be removed.
3. Both sides of the cloth can be used.
4. UCS is gentle, soft and effective debridement.

Before UCS debridement



Before UCS debridement



5 minutes after using 1 x UCS debridement



1 x UCS daily for 10 days



Useful interdigitally





UCS™ Debridement

A pre-moistened cleaning and debridement cloth

UCS is a sterile, pre-moistened debridement cloth that can be used to clean wounds, remove dead skin cells and clean between toes, as well as to moisturise the whole limb. It is a class II B medical device, is CE marked and is available on the Drug Tariff.

Areas of use:

- Interdigital.
- General skin care.
- Perilesional.
- Cleaning of acute, traumatic wounds.
- Ulcer (edge & bed).
- 1st & 2nd degree burns.
- Fistula abscesses.

Features:

- UCS can be used to clean emollient from legs & effectively prepare the wound and leg for healing.
- Both sides of the cloth can be used.
- No water is required.
- UCS has more to offer the skin than water - it contains a surfactant. Surfactants break the surface tension to effectively clean and prepare the wound for healing.
- The mechanical nature of the cloth catches debris as well as facilitating effective debridement.
- On dry skin conditions, the UCS ingredients penetrate the skin cells and softens the skin.
- Reduces pain and increases concordance to treatment.

UCS Debridement is supplied in packs of 10 individual sterile sachets and is available:

- On FP10/GP10 prescription under Part IXA, Physical Debridement
- On the Scottish Wound Management Product contract (NP599/15, ranked number one)
- On the Advanced Wound Care contract for NHS Supply Chain
- Directly from medi UK

Benefits for the patient:

- Soft debridement cloth pre-moistened with a mild cleansing solution which moisturises and softens without damaging healthy cells.
- Odour reduction.
- Enables self-care.
- Cleans and improves skin conditions with no need to soak.
- Safe and simple to use.

Benefits for the clinician:

- Unique, convenient and safe way to clean the leg & wound.
- Treating the leg, treats the person.
- Save time and improve skincare.

Active ingredients: Aloe Vera, Allantoin, Poloxamer (a wetting agent)

Reference:

Downe A. 2014
 'How wound cleansing and debriding aids management and healing'. JCN Vol. 28 No. 4

Order codes:

Product Code	DT500
PIP Code	384-4271
NHS Supply Chain Code	ELZ746
Scottish Contract PECOS Code	217106