

Examples of patient cases mediven® flat knit



Due to the material, the wall stability, the fit and the product features, we recommend mediven flat knit products for patients with the following characteristics*

	Light material / wall stability mediven® mondi®	Medium material / wall stability mediven® cosy	Strong material / wall stability mediven® 550 leg
Lipoedema, stages 1, 2	<p>Lipoedema, stages 1, 2</p> <ul style="list-style-type: none"> • Orthostatic oedema tendency • Uniform thickening of the underlying tissue at certain points • Knot-like structures in thickened underlying tissue • Tendency to bruising • Negative Stemmer's sign <p>Patient-individual criteria</p> <ul style="list-style-type: none"> • No deepened skin folds and shape distortion • No associated weight disorders • Low to moderate pain to touch 	<p>Lipoedema, stages 2, 3</p> <ul style="list-style-type: none"> • Pronounced orthostatic oedema tendency • Knot-like structures in thickened underlying tissue • Pronounced circumferential propagation with overhanging tissue parts • Tendency to bruising • Negative Stemmer's sign <p>Patient-individual criteria</p> <ul style="list-style-type: none"> • Moderate to severe shape distortion** • Associated weight disorders • Low to moderate pain to touch • After liposuction*** 	<p>Lymphoedema, stage 2, 3</p> <ul style="list-style-type: none"> • No spontaneous reversible oedema • Secondary tissue changes • Usually positive Stemmer's sign • Large hard distortion and tissue overhang • Common skin changes associated with lymphoedema <p>Patient-individual criteria</p> <ul style="list-style-type: none"> • Deep skin folds • Associated weight disorders / morbid obesity
CVI with severe oedema tendency	<p>Severe phlebological Indication with oedema tendency</p> <ul style="list-style-type: none"> • Chronic venous insufficiency (CVI) with severe oedema tendency • Recurrence prophylaxis of the leg ulcer venosum (C5) with oedema tendency <p>Patient-individual criteria</p> <ul style="list-style-type: none"> • No deepened skin folds and shape distortion • No associated weight disorders • Little experience with flat knit (transition round knit to flat knit) 	<p>Lymphoedema, stages 1, 2 (stable)</p> <ul style="list-style-type: none"> • Spontaneous reversible lymphoedema or static lymphoedema • Secondary tissue changes • Oedema with soft consistency <p>Patient-individual criteria</p> <ul style="list-style-type: none"> • Light / moderate deepened skin folds • High activity level possible (due to flexible knitted fabric) 	<p>Lipoedema, stages 2, 3; Lipo-Lymphoedema</p> <ul style="list-style-type: none"> • Pronounced orthostatic oedema tendency • Knot-like structures in thickened underlying tissue • Pronounced circumferential propagation with overhanging tissue parts • Tendency to bruising • Positive Stemmer's sign at secondary lymphoedema <p>Patient-individual criteria</p> <ul style="list-style-type: none"> • Grossly deepened skin folds and shape distortion • Associated weight disorders / morbid obesity • Very painful to touch

For flat knit beginners

For easy donning

For highest wall stability



Positive Stemmer's sign
Within the scope of these tests it is not possible to lift a skin fold at the base joint of toes (left in the picture, right normal).

* Please note that this recommendation is neither the diagnosis of the healthcare staff, nor the consultation by the medical specialized trade replaced. The choice of the optimal compression stocking depends on the indication and the individual needs of the patient. The most prominent condition indicates stocking choice.

** medi tip: The high longitudinal elongation of the knitted fabric in the mediven cosy can make it easier to put on, for example in the presence of shape distortion.

*** medi recommendation: if the symptoms of lipoedema persist, for example oedema or pressure pain.

Examples of patient cases mediven® flat knit

Light material / wall stability

For flat knit beginners

mediven® mondi®

Product features

- Light wall stability
- High lateral elongation
- Flexible, supple fabric

Special features

- Ideal for pressure sensitive patients
- Ideal for patients changing from round knit to flat knit
- Incorporation of silver segments possible (for an antibacterial effect)

Medium material / wall stability

For easy donning

mediven® cosy

Product features

- Medium wall stability
- High longitudinal elongation
- Flexible, supple fabric

Special features

- Maximum freedom of movement
- Ideal for an active everyday life
- Ideal before or after liposuction**

Strong material / wall stability

For highest wall stability

mediven® 550 leg

Product features

- Highest wall stability, high working pressure
- Low lateral and longitudinal elongation
- Firm, strong fabric

Special features

- Highest wall stability for ideal oedema reduction
- Possibly experienced subjective pain relief (e.g. in the case of lipoedema)

** medi tip: The high longitudinal elongation of the knitted fabric in the mediven cosy can make it easier to put on, for example in the presence of shape distortion.