



Helping you to help yourself. Wound cleaning and skin care for use at home The unique pre-moistened UCS™ cloth

The UCS debridement cloth is a pre-moistened single use medical device for wound and skin cleansing. Wound cleaning at an early and appropriate stage is likely to accelerate wound healing and improve patient care.

The solution within the UCS will soften any yellow tissue and debris that halts wound healing. This then allows the unique loops of the fabric to catch and hold debris and effectively remove it from the wound bed. The skin friendly cleanser (surfactant) within the solution will disrupt and remove unnecessary bacteria from the wound and surrounding area.

UCS can also be used to clean and moisturise dry skin.

This is a cleansing cloth and is not to be used as a dressing—initial use should be under clinical guidance

The combination of the cloth and solution is all that is required. No water or saline needs to be added. Allow the solution to dry on the skin, do not towel dry or wash off.

To use UCS:

- ◆ Open sachet to access the pre-moistened UCS cloth
- ◆ Use this to clean the wound with a gentle polishing motion
- ◆ Once the wound has been cleaned the other side of the UCS can be used to clean the surrounding skin.
- ◆ Do not use the same cloth for both legs (risk of cross infection)

**Available on prescription from your GP in packs of ten,
order code DT500 PIP code 384-4271**

