

# medi Hospitalbutler

The patented donning aid for use in day-to-day hospital care and in the operating theatre

## Properties

- Simple handling
- Time-saving and economical
- Pain-free donning of clinical compression stockings for the patient
- Can also be used for lower leg injuries



Adjustable diameter, hard paint finish, white

## Simple donning



1. By lifting the locking device the diameter of the medi Hospitalbutler can be adjusted so that it can be pulled up over the patient's ankle, or calf if necessary, without contact.



2. The topband is gathered up over the stirrup and the stocking pulled up.



3. The stocking is pulled up until the colour mark appears at the upper edge of the stirrup.



4. The medi Hospitalbutler is pushed up over the ankle and – if possible – up to below the patient's knee. The medi Hospitalbutler is pulled off downwards and the stocking pulled up to the inguinal fold.



5. The tension determines the length of the stocking.

## Material composition:

powder-coated steel tubing

See page 33 for the article numbers.